


IntelViewer Install Instructions--Windows

- Please click on the below link to obtain the Windows PC installer file.


[San Radiology & Nuclear Medicine: IntelPACS: IntelViewer Tracks](#)

Use the Stable Track, click download.

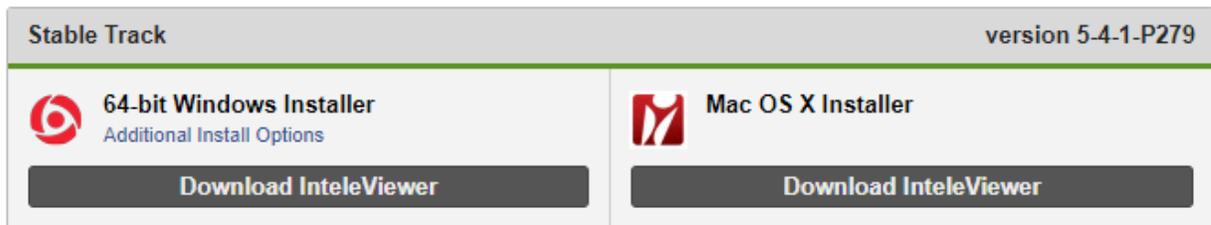


The screenshot shows a download interface for IntelViewer. At the top, it says "Stable Track" on the left and "version 5-4-1-P279" on the right. Below this, there are two columns. The left column features the IntelViewer logo, the text "64-bit Windows Installer" with a link for "Additional Install Options", and a dark grey button labeled "Download IntelViewer". The right column features a Mac OS X logo, the text "Mac OS X Installer", and a dark grey button labeled "Download IntelViewer".


- Browse to your downloads folder to access the installer:

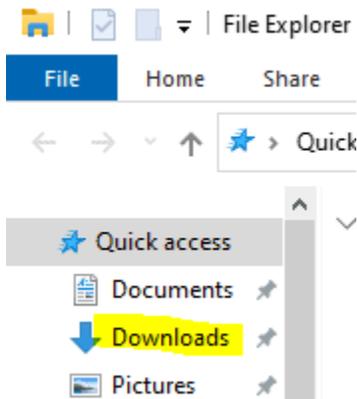

- Double click on the installer file:

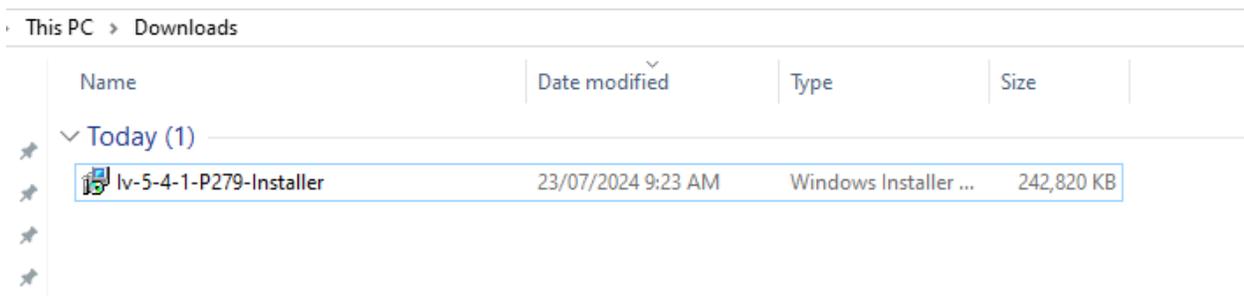

- Follow the prompts in the installer and leave the default settings. Keep hitting next and accept to all the prompts.

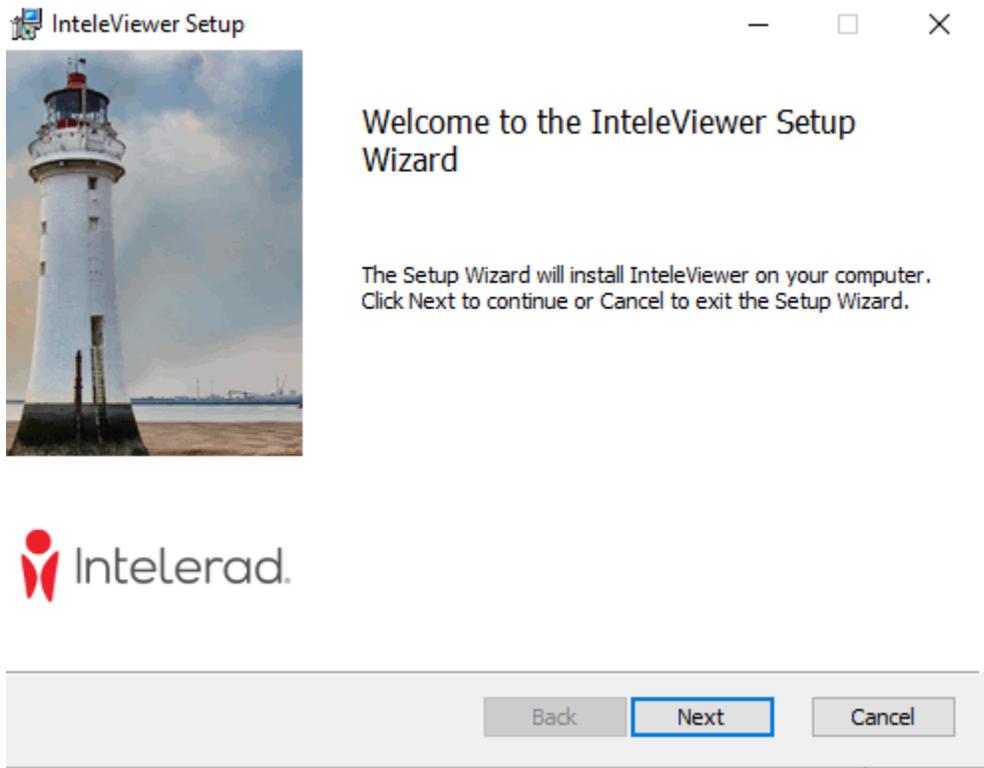

- One the installer finishes the icon should be on the desktop. Please double click on the icon:



- The first time you open IntelViewer, it will ask you to enter a PAC's bookmark
(For the server URL please enter: <https://pacs.sanradiology.com.au/>)





inteleViewer™

Server Bookmarks

+ Add **✎** Edit

	San Radio	s
		m.au

Inte
Versio
Part

EN

Close this session

Server Bookmarks

Edit Bookmark

Server URL:

Protect Bookmark

Use Custom Server Name

Name:

Advanced

Ok **Cancel**

Close

- You should then be able to log in with your PAC's Username and password:

